SIEMENS

easyPocket™

ProPocket™

Remote Control

User Guide

usa.siemens.com/hearing

Life sounds brilliant.

Congratulations!

Dear Customer,

Your hearing instruments are equipped with technology that allows them to be controlled by your remote control.

These instructions describe how to operate your remote control quickly and easily.

It also provides you with important information on how to handle your instruments.

Your Hearing Care Professional can answer any further questions about instructions and proper use.

Enhance your Siemens experience.

Learn about your hearing aids and accessories you may find helpful, access the latest information about hearing health, and receive promotional offers by joining the Siemens hearing health community at*:

<http://hearing.siemens.com/us/en/services/subscription/subscribe.html>

*We respect your privacy. We will not sell, rent, or give your name or e-mail address to anyone. At any time, you can select a link at the bottom of every email to unsubscribe or update profile.


Table of contents

easyPocket™ remote control	3
Key lock	6
Batteries	7
Daily use	8
Alarm clock	11
Setup and service functions	13
Propocket™ remote control	16
Changing the batteries	17
Key lock	17
Turning your hearing instruments on and off	18
Adjusting the volume	19
Changing the program	20
Safety information	21
Maintenance and care	24
Technical information	25
Performance checklist	27
Warranty and service	28

easyPocket remote control


The operating distance of the easyPocket remote control is approximately 40 inches (1 meter).

- Make sure the distance between your remote control and hearing instrument does not exceed the operating distance.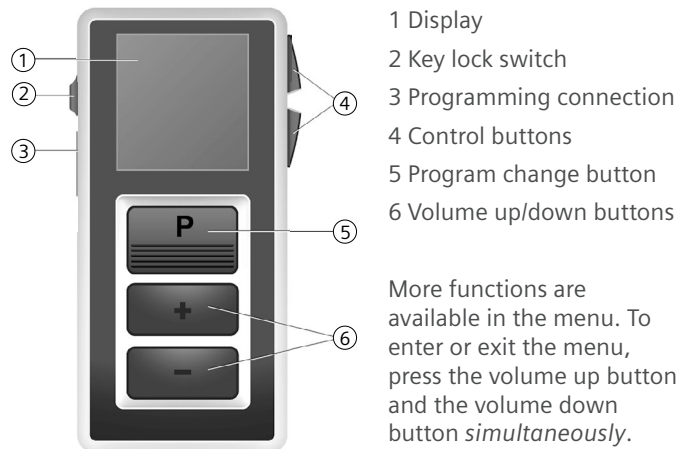


Display

All actions initiated with the remote control are immediately reflected on the display. For example, if you press the volume up button, the following icon is displayed.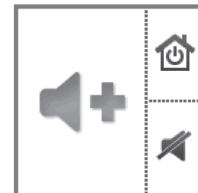


The remote control does **not** receive information from your hearing instruments. All actions that you initiate with the controls of your hearing instruments are **not** reflected on the remote control's display.

Display of easyPocket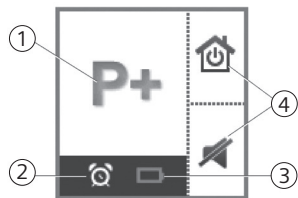


- 1 Indicates change of hearing program and volume
- 2 Alarm clock is activated
- 3 Batteries of the remote control are low
- 4 Function of control buttons

Additional display options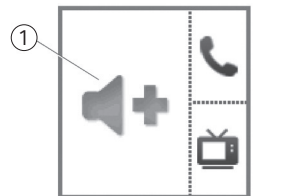

- 1 Indicates change of volume
- 2 Program name
- 3 Program icon
- 4 Program number
- 5 Alarm clock is activated
- 6 Batteries of the remote control are low
- 7 Function of programmable control buttons

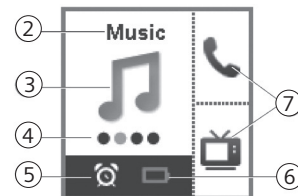Power save mode

If the remote control is not used for some time, the display automatically turns black. The remote control is then in its power save mode.

- Press any button to activate the display again.

Control buttons

You can quickly access two functions with the control buttons on the right hand side of the remote control. The control buttons of the easyPocket can be configured by your Hearing Care Professional. Small icons on the display show the functions.

Control functions	
	Turn hearing instruments on or off.
	Reset to default volume and hearing program
	Tinnitus therapy feature level up.
	Tinnitus therapy feature level down

Adjust treble

Adjust bass

All other icons:

Shortcut to select a preferred hearing program.

The available icons and hearing programs depend on your personal setup.

Key lock

Key lock switch

To prevent any accidental operation while the remote control is in pockets or bags, move the key lock switch to lock position (red color visible).

When the key lock is active, all buttons on the remote control are disabled. The following icon is displayed.

Programming connection

This connection is only to be used by your Hearing Care Professional for programming the remote control.

Batteries

The remote control uses two AAA batteries.

Inserting batteries

- Slide the battery compartment cover in the direction of the arrow.

Insert the batteries.

Make sure the “+” symbols on the battery and compartment are aligned.

- Attach the cover by sliding it onto the remote control.

The current time setting is displayed. If you want to change it, refer to section “Setting the time” (page 13).

Checking the battery status


When the batteries of the remote control are low, this is indicated in the status bar.

Additionally, you can check the battery status at any time in the menu.

- To enter the menu, press the volume up button and the volume down button *simultaneously*.
- Navigate to the menu entry Info by pressing the volume up button or the volume down button.
- Confirm your selection by pressing the program change button.

The battery status is displayed:

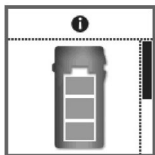battery full

battery empty

- To return to the standard display, press the program change button.

Remove empty batteries immediately and dispose of them according to local regulations.

Daily use

Adjusting the volume

If you are wearing two hearing instruments, the volume will be adjusted on both instruments at the same time.



- Press the volume up button or the volume down button to adjust the volume by one step.

Changing the hearing program

If you are wearing two hearing instruments, the hearing program will be changed on both instruments.

- Press the program change button to switch to the next hearing program.

Your Hearing Care Professional may configure your easyPocket to show specific situations.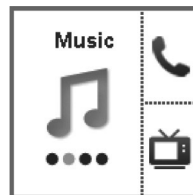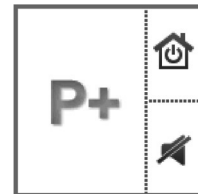

- Press the program change button repeatedly until the icon of the desired hearing program is displayed.


or

- Press the control button next to the icon indicating the desired hearing program.

The control buttons have to be configured by your Hearing Care Professional.

Muting

You have several options to mute your hearing instruments with your remote control. Both hearing instruments are simultaneously muted.

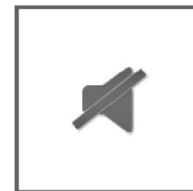When you mute your hearing instruments via remote control, the following icon is displayed. It is displayed until you unmute your hearing instruments.

Additionally, all buttons are disabled. The only available function is to unmute the hearing instruments.

After unmuting, the previously used volume and hearing program are set.

Via control button

Press the control button next to the following icon to mute or unmute your hearing instruments. The control buttons have to be configured by your Hearing Care Professional.

Via menu

Mute:

- To enter the menu, press the volume up button and the volume down button *simultaneously*.
- Navigate to the menu entry **HI Mute / Unmute** by pressing the volume up button or the volume down button.
- Confirm your selection by pressing the program change button.

Unmuting:

- Press any button to activate the display.
- Wait until the information about the muted hearing instruments disappears.

The menu is displayed and the **HI Mute / Unmute** entry is still selected.

- To unmute your hearing instruments, confirm the selection by pressing the program change button.

Adjusting the tinnitus therapy feature level

This function is only available if your Hearing Care Professional has configured the tinnitus therapy signal for both your hearing instruments and for the control buttons of the remote control.

You can recognize the setting, if the corresponding icons are displayed next to the control buttons.

- To adjust the tinnitus therapy feature level, press the upper or the lower control button.

Alarm clock

You can set an alarm clock to remind you at the same time every day by a repeating signal tone and by an alarm clock icon.

The alarm is initiated by the remote control but the acoustic signal is emitted by your hearing instruments.

- Turn on your hearing instruments.
- Keep your hearing instruments within the remote control's operating distance. Otherwise you will not be able to hear the alarm.

Setting the alarm clock

- To enter the menu, press the volume up button and the volume down button *simultaneously*.
 - Navigate to the menu entry **Alarm** by pressing the volume up button or the volume down button.
 - Confirm your selection by pressing the program change button.
- The entry field for hours is highlighted.
- Set the hours by pressing the volume up button or the volume down button.

Confirm your setting by pressing the program change button.

The entry field for minutes is highlighted.

- Set the minutes by pressing the volume up button or the volume down button.
- Confirm your setting by pressing the program change button.

The alarm on/off selection field is highlighted.

- Press the volume up button or the volume down button to position the cursor next to alarm clock on:

- Confirm your selection by pressing the program change button.

When the alarm clock is turned on, this is indicated in the status bar.

Ending the alarm

When the alarm clock rings, a corresponding icon is displayed on the remote control.

If you do not end the alarm, it will ring again a few minutes later.

- To end the alarm, press the program change button. The alarm clock will ring at the same time on the next day.

Turning the alarm clock off

- To enter the menu, press the volume up button and the volume down button *simultaneously*.
- Navigate to the menu entry **Alarm** by pressing the volume up button or the volume down button.
- Confirm your selection by pressing the program change button.

The entry field for hours is highlighted.

- Press the program change button twice to skip the entry fields for hours and minutes.

The alarm on/off selection field is highlighted.


- Press the volume up button or the volume down button to position the cursor next to alarm clock off: 
- Confirm your selection by pressing the program change button.

Setup and service functions

Setting the time

- To enter the menu, press the volume up button and the volume down button *simultaneously*.
- Navigate to the menu entry **Clock** by pressing the volume up button or the volume down button.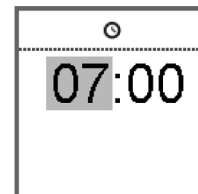

- Confirm your selection by pressing the program change button.

The entry field for hours is highlighted.

- Set the hours by pressing the volume up button or the volume down button.
 - Confirm your setting by pressing the program change button.
- The entry field for minutes is highlighted.
- Set the minutes by pressing the volume up button or the volume down button.
 - Save the settings by pressing the program change button.

The time is displayed.

The time is always displayed in the header of the menu. To enter or exit the menu, press the volume up button and the volume down button *simultaneously*.

Resetting volume and hearing program

If you have made changes to volume or program and do not know why the hearing instruments are acting the way they are, you can manually set the **default** volume and hearing program for your hearing instruments.

- To enter the menu, press the volume up button and the volume down button *simultaneously*.
- Navigate to the menu entry **HI Restore** by pressing the volume up button or the volume down button.
- Confirm your selection by pressing the program change button.

The hearing instruments are reset and a corresponding icon is displayed.

- You can also reset the hearing instruments by pressing the control button next to the following icon.

The control buttons have to be configured by your Hearing Care Professional.

Displaying contact data

If your Hearing Care Professional has saved his or her contact data on the remote control, you can view this information:

- To enter the menu, press the volume up button and the volume down button *simultaneously*.
- Navigate to the menu entry **Contact** by pressing the volume up button or the volume down button.
- Confirm your selection by pressing the program change button.

The contact data is displayed.


- To return to the standard display, press the program change button.

Setting the language of the menu

- Ask your Hearing Care Professional.


ProPocket remote control

ProPocket™ Remote Control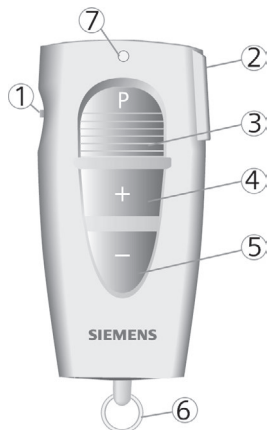

- 1 Lock switch
- 2 On/Off of hearing instruments
- 3 Program change
- 4 Volume up
- 5 Volume down
- 6 Accessory ring
- 7 LED


Changing the batteries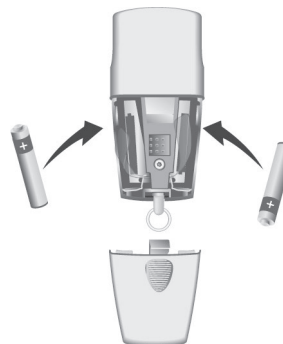

The ProPocket remote control is powered by two AAAA batteries.

- The battery compartment cover is located on the back of the remote control. Press the ridged indentation with your thumb in order to slide the cover off.
- Insert two batteries as shown in the picture. Make sure the polarity symbols on the battery and compartment are aligned
- Slide the battery cover back in place. A click is audible when the battery compartment is fully closed.

Key lock

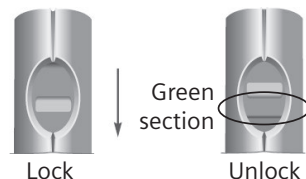The lock switch deactivates the buttons on the remote control so that accidentally pressing the buttons will not have any effect.

- Slide the control over so the green section is covered. The lock is activated. When pressing a button the LED blinks 3 times to indicate that the lock is activated.

- Slide the control in the other direction so the green section shows.

The lock is deactivated.

Turning your hearing instruments on and off

On/Off Button

- Make sure the distance between your remote control and hearing instrument does not exceed 40 inches (1 meter).
- If you are wearing two hearing instruments, both instruments will be turned on/off at the same time.

- Press the On/Off button to turn your hearing instruments on
- Press the On/Off button to turn your hearing instruments off

Adjusting the volume

- Make sure the distance between your remote control and hearing instrument does not exceed 40 inches (1 meter).
- If you are wearing two hearing instruments, the volume will be changed on both instruments at the same time.

Press the "Volume up" (+) button to increase the volume of your hearing instruments.

Volume up

Press the "Volume down" (-) button to decrease the volume of your hearing instruments.

Volume down

Your Hearing Care Professional can program your hearing instruments so that you hear acoustic signals to indicate when the volume has been changed and when the volume limit has been reached.

Changing the program

- Make sure the distance between your remote control and hearing instrument does not exceed 40 inches (1 meter).
- If you are wearing two hearing instruments, the program will be changed on both instruments at the same time.

Each time you press the program button, your hearing instruments switch to the next program, confirming each change with an acoustic signal (if programmed this way by your Hearing Care Professional).

Program button

Example: The hearing instruments are in program 1. You want to change to program 2.

- Press the program button once. You will hear an acoustic signal. You are now in program 2.

Safety information

⚠ WARNING

Wireless systems may interfere with measuring devices and electronic equipment.

- Do not use remote control in hospitals or airplanes.
- Have the key lock activated in hospitals or airplanes.
- Have the ePen tip closed in hospitals or airplanes.

⚠ WARNING

Pacemaker compatibility

Siemens remote control devices have been tested and found not to cause interference with cardiac pacemakers that are manufactured and operated in compliance with applicable safety standards, provided a minimum distance of 12 inches (30cm) is maintained between the remote control and the pacemaker. To accommodate this provision:

- Do not activate the remote within 12 inches of the pacemaker.
- Do not carry the remote in your breast pocket.
- Do not wear the remote with a lanyard around your neck.
- Do not place remote directly on the skin over the implanted pacemaker.

Following these precautions, Siemens remote controls can be operated safely by, or within the vicinity of, individuals fitted with these pacemakers.

- For all active implants or life supporting systems other than a pacemaker: prior to use, have the electromagnetic compatibility verified.

WARNING

Choking hazard posed by small parts.

- Keep hearing instruments, batteries and accessories out of children's reach.
- If swallowed, consult a physician or hospital immediately.
- If infants, small children or mentally disabled persons need to wear hearing instruments, ensure adequate supervision.

Battery tips

CAUTION! Batteries can be harmful if swallowed.

- If a battery is accidentally swallowed, seek medical attention immediately, or call **The National Button Battery Hotline collect at (202) 625-3333**.
- Always use the battery size recommended on your warranty card or by your Hearing Care Professional for your model for best performance.
- Remove the battery from your instrument when not in use. This prevents possible damage from expansion or leakage of the battery, if the instrument is not fully turned off.
- Always carry spare batteries in the event that your current battery goes dead. Place each spare battery in an individual, non-metallic container to prevent contact with other batteries or metal objects, such as keys or coins.
- Follow your local recycling rules for battery disposal.

Note: Do not throw used batteries into household trash. Return them to your Hearing Care Professional for environmentally correct disposal.

WARNING

Risk of explosion!

- Do not use your remote control in explosive atmospheres (e. g. in mining areas).

NOTICE

Your hearing instruments are designed to comply with international standards on electromagnetic compatibility but interference with nearby electronic devices could occur. In this case, move away from the source of interference.

NOTICE

For instruments with wireless connectivity: When the telecoil program is active, the remote control may cause a pulsing noise.

- Use the remote control with a distance greater than 4 inches (10 cm).

NOTICE

- Your remote control is programmed to exclusively communicate with your hearing instruments. If you experience problems with another person's remote control, consult your Hearing Care Professional.

Maintenance and care

Protection

Use the “cover” to protect against accidental drips or splashes when you go near water such as a sink.

Do not let your hearing instrument get wet.

Cleaning

Note: Do NOT put your remote control in water! The cover does NOT make your remote control waterproof. Do NOT clean your remote control with alcohol or benzene.

- Clean the remote control as necessary with a soft tissue.
- Avoid abrasive cleaning liquids with additives such as citric aids, ammonia, etc.

Note: The remote control is sensitive to extreme heat, high humidity, and mechanical stress. Do not expose the remote control to extreme temperature or high humidity. Do not leave it in direct sunlight. Do not immerse it in water. Do not dry it off in a microwave oven.

Do not expose the remote control to strong radiation, e.g. X-Ray or MRI examinations. Weaker radiation, e.g. from airport security or radio equipment will not damage the device.

Technical information

Wireless technology provides synchronization between two hearing instruments. If you change the program or volume on one instrument, the other is automatically changed. In addition, wireless technology automatically adjusts the system to change both hearing instruments simultaneously to adapt to different listening situations.

Operating frequencies ProPocket: Fc=3.28 MHz

CE, FCC and IC compliance information can be found inside the battery compartment.

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Siemens easyPocket

FCC: SGI-WL700

IC: 267AB-WL700

ProPocket remote control

FCC: SGI-WL101

IC: 267AB-W101

This Class B digital apparatus II radiocommunication device complies with Part 15 of the FCC rules and Industry Canada Standard RSS-210. Operation is subject to the following two conditions:

- (1) this device may not cause interference.
- (2) this device must accept any interference, including interference that may cause undesired operation of the device.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC rules and Industry Canada Standard RSS-210. These

limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Performance Checklist

Problem	Possible Solution
The remote control does not work	<ul style="list-style-type: none"> • Check if the key lock is locked. Unlock it. • Check if batteries are empty. Change the batteries. • Move away from the electro-magnetic noise sources.
Another remote control affects your hearing instruments	<ul style="list-style-type: none"> • Return remote control and hearing instruments to your Hearing Care Professional.

If you still encounter problems after trying these suggestions consult your Hearing Care Professional.

Warranty and service

Your remote control has a one-year warranty that covers defects in material and workmanship from the original date of purchase. There is no loss and damage coverage for the remote control. Remotes received for warranty service at an authorized Siemens Service Center will be repaired or replaced with new or reconditioned components, without charge, to meet the performance specifications of the product.

Warranty service must only be performed by an authorized Siemens Service Center. Service performed by unauthorized service entities voids this warranty; and repairs so necessitated will be done on a parts and labor cost basis.

This warranty does not cover malfunctions due to unusual wear and tear or mistreatment of the product such as physical shock or tampering with the product, any of which voids all warranties.

In case of malfunction or loss, take your Siemens remote control and proof of purchase to the Hearing Care Professional from whom the unit was purchased. If factory service or replacement is needed, your Hearing Care Professional will forward the unit to an authorized Siemens Service Center.


IMPORTANT

Record your remote control serial number and date of purchase here for future reference.

Serial number

Date of purchase

The serial number is located inside the battery compartment.

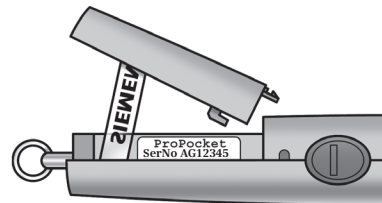The year of manufacture is incorporated into the serial number. The warranty is based upon the date of purchase, not the date of manufacture.

The serial number may look similar to this sample: AG12345. The second letter of this serial number refers to the year of manufacture of the remote control. The letters D, E, and F indicate 2013 year of manufacture, while G and H indicate 2014. Your Hearing Care Professional can answer any questions you may have about the identifying code on your remote control.

Code (2nd Digit)	Year
Y, A	2011
B, C	2012
D, E, F	2013
G, H	2014

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Information in this brochure is subject to change without notice. Copyright © 2015 Siemens AG. All rights reserved.
La información que contiene este folleto está sujeta a cambios sin aviso. Copyright © 2015 Siemens AG. Todos los derechos reservados.
under Trademark License of Siemens AG
Sivantos GmbH
Sivantos, Inc.