 **WSAUD A/S** Nymoellevej 6, DK-3540 Lyngby, Denmark
<https://global.widex.com>

 **Manual no.:** 9 514 0313 080 #07
Issue: 2020-11

9 514 031 308 000





COM-DEX with firmware 1.3 and newer

DEX™
A WIRELESS WORLD

COM-DEX

GB
DE
FR
ES
DK
IT
JP
US

WIDEX®
HIGH DEFINITION HEARING

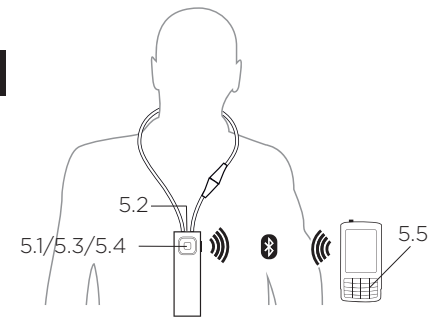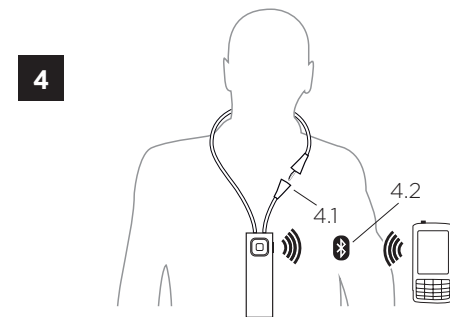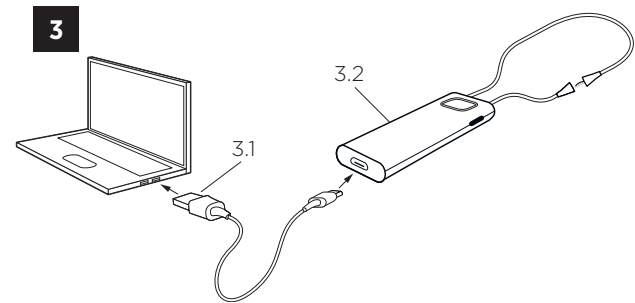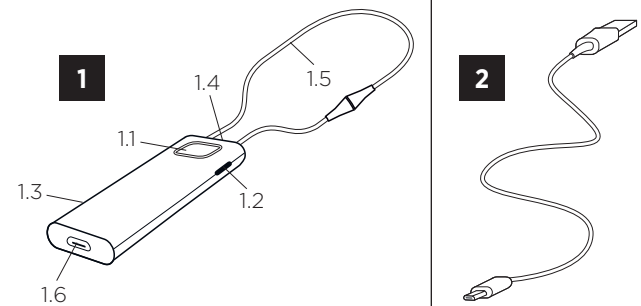GB	USER INSTRUCTIONS	2
DE	BEDIENUNGSANLEITUNG.....	14
FR	USER INSTRUCTIONS	26
ES	INSTRUCCIONES DE USO	39
DK	BRUGSANVISNING.....	52
IT	ISTRUZIONI PER L'USO	64
JP	取扱説明書.....	77
US	USER INSTRUCTIONS	90
	FCC AND ISED STATEMENTS	118

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Intended use

The device is intended to improve the ability of hearing impaired people to conduct mobile phone conversations while using their hearing aids, to improve the usability and ease of control of such hearing aids by equipping them with a Smartphone-based control method.

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2.1 USB cable

CHARGING – *See illustration 3*

- 3.1 Connect USB cable between the device and the power supply. We recommend that the two neck loop parts are not joined while charging the device.
- 3.2 Red light: charging, green light: fully charged.

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How to pair the device with your phone

To pair your device with your phone for the first time, follow these steps:

1. Make sure that the device and phone are within 1 metre range of each other.
2. Connect the neck loop. The device will blink red and blue.
3. Activate Bluetooth on your phone.
4. Start the pairing process according to the instructions in your phone's user guide.
5. Your phone will show a list of Bluetooth devices.
Select COM-DEX from the list.

Once you have paired the device, it will remain so even when you disconnect the neck loop.

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Note: Answering and ending calls is also possible on the mobile phone. Note that, on some mobile phones, you may need to select the COM-DEX from a list of devices. Otherwise the sound will be transmitted through the phone's speakers.

Outgoing calls – *See illustration 5*

- 5.5 Dial from phone.

Note: You can also use the device to stream sound from any sound source to your hearing aids.

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To shut out surrounding sounds: press Room Off button – See *illustration 1*.

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You can download the COM-DEX app from the App Store or Google Play. This app allows you to use your phone as a remote control.

MAINTENANCE

- Clean with a damp cloth. Do not use chemicals.
- Do not expose to extreme temperatures or high humidity.
- Do not immerse in liquids.
- When the device is not in use keep it in a dry location out of reach of children and pets.
- Never try to open or repair the device yourself. This may only be done by authorised personnel.

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For further information, especially regarding patients with medical implants, download the document: “Information on Widex-Link™” at <https://global.widex.com/widexlink>

TROUBLESHOOTING

Problem	Possible cause	Solution
The device does not work	a) Device battery low b) The mobile phone is set to silent mode c) Hearing aid battery low d) Out of range	a) Charge device battery b) Set to normal mode c) Change hearing aid's battery d) Bluetooth has a range up to 10 m.
No sound	a) Not connected via Bluetooth	a) Repeat the pairing process

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N26346

SYMBOLS

Symbols commonly used by WSAUD A/S in medical device labelling
(labels/IFU/etc.)

Symbol	Title/Description
	Manufacturer The product is produced by the manufacturer whose name and address are stated next to the symbol. If appropriate, the date of manufacture may also be stated.
	Date of manufacture The date when the product was manufactured.
	Batch code The product's batch code (lot or batch identification).
	Catalogue number The product's catalogue (item) number.
	Serial number The product's serial number.*
	Consult instructions for use The user instructions contain important cautionary information (warnings/precautions) and must be read before using the product.

Symbol	Title/Description
	<p>Caution/Warning Text marked with a caution/warning symbol must be read before using the product.</p>
	<p>WEEE mark “Not for general waste” When the product is to be discarded, it must be sent to a designated collection point for recycling and recovery.</p>
	<p>CE mark The product is in conformity with the requirements set out in European CE marking directives.</p>
	<p>RCM mark The product complies with electrical safety, EMC and radio spectrum regulatory requirements for products supplied to the Australian or New Zealand market.</p>
	<p>Interference Electromagnetic interference may occur in the vicinity of the product.</p>

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DESCRIPTION OF DEVICE – *See illustration 1*

- 1.1 Push button
- 1.2 Room Off button
- 1.3 LED
- 1.4 Microphone
- 1.5 Neck loop
- 1.6 Micro USB

Note: Read these instructions carefully before you start using the device.

Intended use:

This device is intended to improve your ability to use Bluetooth enabled mobile phone connections with your hearing aid(s). The aid(s) can further be manipulated by the controls on the mobile phone.

Note: The device uses Bluetooth standard which is common in newer mobile phones.

ACCESSORIES – *See illustration 2*

- 2.1 USB cable

CHARGING – *See illustration 3*

- 3.1 Connect USB cable between the device and any power supply.
We recommend that the two neck loop parts are not joined while charging the device.
- 3.2 Red light: charging, green light: fully charged.

Battery life: 8 hours streaming, 8 days standby.

HOW TO USE – *See illustration 4*

- 4.1 Join the two neck loop parts. The neck loop functions as an on/off switch.
- 4.2 Pair the device to your mobile phone using Bluetooth.

How to pair the device with your mobile phone

To pair your device with your mobile phone for the first time, follow these steps:

1. Make sure that the device and mobile phone are within 3 ft range of each other.
2. Connect the neck loop. The device will blink red and blue.
3. Activate Bluetooth on your mobile phone.
4. Start the pairing process according to the instructions in your mobile phone's user guide.
5. Your mobile phone will show a list of Bluetooth devices.
Select COM-DEX from the list.

Once you have paired the device, it will remain paired even when you disconnect the neck loop.

Incoming calls – *See illustration 5*

- 5.1 Answer call: Press push button.
- 5.2 Speak into microphone while device is hanging around your neck.
- 5.3 End call: Press push button.
- 5.4 Reject call: Long press on push button.

Note: Answering and ending calls is also possible on the mobile phone. Note that, on some mobile phones, you may need to select the COM-DEX from a list of devices. Otherwise the sound will be transmitted through the mobile phone's speakers.

Outgoing calls – *See illustration 5*

- 5.4 Dial from mobile phone.

Note: You can also use the device to stream sound from any Bluetooth enabled sound source to your hearing aids.

Sound adjustment during use

Volume is adjusted on the mobile phone.

To shut out surrounding sounds: press Room Off button – *See illustration 1.*

Note: If you are in a particularly noisy environment or if you need to increase the volume to a higher level than normal while streaming sound to your hearing aids, keep the Room Off button pressed for 5 seconds. To return to your normal sound level, press the Room Off button again and keep it pressed for 5 seconds.

You can download the COM-DEX app from the App Store or Google Play. This app allows you to use your mobile phone as a remote control.

MAINTENANCE

- Clean with a damp cloth. Do not use chemicals.
- Do not expose to extreme temperatures or high humidity.
- Do not immerse in liquids.
- When the device is not in use keep it in a dry location out of reach of children and pets.
- Never try to open or repair the device yourself. This may only be done by authorized personnel.

INFORMATION ABOUT EMISSION

Emission levels from Widex devices are several times lower than the levels transmitted by mobile phones and other communications equipment.

For further information, especially regarding patients with medical implants, download the document: “Information on WidexLink™” at <https://global.widex.com/widexlink>

TROUBLESHOOTING

Problem	Possible cause	Solution
The device does not work	a) Device battery low b) The mobile phone is set to silent mode c) Hearing aid battery low d) Out of range	a) Charge device battery b) Set to normal mode c) Change hearing aid battery d) Bluetooth has a range up to 30 ft.
No sound	a) Not connected via Bluetooth	a) Repeat the pairing process

If the problems persist, contact your hearing care professional.

WARNINGS

- Do not leave the device unattended while charging.
- Your device should be stored and transported within the temperature and humidity ranges of -10° to +50°C (14°F to 122°F) and <90% RH.
- Your device is designed to operate from 5°C (41°F) to 40°C (104°F).
- Keep the device protected from dust or external particles.
- We recommend using a 5V up to 500 mA USB adaptor.
- Do not use the device in mines or other areas with explosive gases.
- When the device is connected via micro USB cable to external mains-operated equipment such as a charger, PC, audio source or similar, this equipment must comply with IEC 60950-1, IEC 62368-1, IEC 60601-1 or equivalent safety standards.
- Only the accessories and cable that are included in the sales box must be used.
- Using cables other than those specified or provided may result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

CONDITIONS FOR USE

Operate the device within the following conditions:

	Minimum	Maximum
Temperature	5°C (41°F)	40°C (104°F)
Humidity	10% rH	90% rH
Atmospheric pressure	750 mbar	1060 mbar

Store and transport the device within the following conditions:

	Minimum	Maximum
Temperature	-10° (14°F)	50°C (122°F)
Humidity	10% rH	90% rH

You can find technical datasheets and additional information on your device on <https://global.widex.com/doc>

REGULATORY INFORMATION

Directive 2014/53/EU

WSAUD A/S hereby declares that this COM-DEX is in compliance with the essential requirements and other relevant provisions of Directive 2014/53/EU.

The device contains radio transmitters operating at: 10.6 MHz, -54 dB μ A/m @10 m, 2.4 GHz, 0.9 mW EIRP.

A copy of the Declaration of Conformity can be found at:
<https://global.widex.com/doc>

N26346

Wireless functions and operating characteristics

The following tables summarizes the technical details of the wireless technology:

Technical details	Inductive radio	Bluetooth (LE)
Antenna type	Integral coil antenna	Inductive antenna
Antenna dimensions	Ø 2 mm L: 5 cm	Ø1.8 mm x 4.5 mm
Modulation	FSK	FSK
Magnetic Field Strength	-54dB μ A/m @ 10 m	-54dB μ A/m at 10m distance
Output power (EIRP)		0 dBm / 1mW
Range	0.3 m	6 m
Center frequency	10.6MHz	2401 MHz - 2480 MHz
Channels	Single channel	40

Bandwidth	660kHz	2 MHz
Data rate	212 kbps	1 Mbps
Data flow	Simplex/Semi Duplex	Duplex
Protocol	Random Access – Proprietary	BLE

Wireless Security Measures

Security of the wireless signals is assured through device system design that includes:

- Individual MAC address for each unit which is checked during each transmission.
- A built-in pairing table which specifies valid and legitimate pairing among units
- A proprietary Widex communication protocol which checks the package numbers during each transmission.
- A Cyclic Redundancy Check (CRC) to check data validity and correct errors.

Electromagnetic emissions/immunity standards

The product complies with the following EMC/EMI standards:

Standard	Test type	Note
47 CFR Part 15, Subpart B	RF emissions	US FCC requirements for unintentional radiators
47 CFR Part 15, Subpart C	RF emissions	US FCC requirements for intentional radiators
47 CFR Part 18	RF emissions	US FCC requirements for Industrial, Scientific and Medical Equipment

IEC 60601-1-2	General EMC including RF emissions, RF immunity ESD	Medical Electrical Equipment, Electromagnetic compatibility
IEC 60118-13	RF immunity	International standard for hearing instruments to ensure adequate immunity radio interference from mobile telephones

The following Table summarizes the technical details of the Bluetooth and WidexLink technology as it is implemented in the COM-DEX.

Unit	COM-DEX		Hearing aid
Type	W-link	Bluetooth*	W-link
Antenna type	Inductive Loop antenna	Embedded ceramic antenna	Inductive antenna
Antenna dimensions	L: 70 cm	NA	Ø1.8 mm L – 4.85mm
Modulation	FSK	FHSS/GFSK, $\pi/4$ DPSK, 8DPSK	FSK

Magnetic Field Strength	-26dB μ A/m at 10m distance	NA	-54dB μ A/m at 10m distance
Output power	1.2 nW EIRP** -59 dBm	2.5 mW EIRP +4dBm	29pW EIRP** -75 dBm
Range	<1m remote unit to hearing aid	<10m between COM-DEX and Bluetooth device	<1m remote unit to hearing aid <30cm between hearing aids
Center frequency	10.6MHz	2.45GHz ISM	10.6MHz
Channels	Single channel radio	80 channels	Single channel radio

Bandwidth	660kHz (-15dB)	1MHz	660kHz (-15dB)
Duty Cycle ***	Up to 100 %	< 100 %	< 5 %
Data rate	212kbit/s raw channel capacity	3 Mbit/s	212kbit/s raw channel capacity
Data flow	Simplex or semiduplex	Time division duplex (TDD)	Simplex or semiduplex
Protocol	Random access - no collision avoidance	Packet based protocol, time divided	Random access - no collision avoidance

** Bluetooth specification v 4.1 + EDR/LE published by the Bluetooth Special Interest Group (SIG)*

*** EIRP = Equivalent isotropically radiated power*

**** Duty Cycle averaged over 1 hour of operation*

Cables and transducers:

The COM-DEX is using a micro USB cable for charging.

Guidance and manufacturer's declaration

Electromagnetic emissions

The COM-DEX is intended for use in the electromagnetic environment specified below. The customer or the user of a COM-DEX should assure that it is used in such an environment.

Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	The COM-DEX uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.

RF emissions CISPR 11	Class B	The COM-DEX is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not applicable *)	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Not applicable *)	

*) *Battery powered equipment*

Electromagnetic immunity

The COM-DEX is intended for use in the electromagnetic environment specified below. The customer or the user of a COM-DEX should assure that it is used in such an environment.

Immunity Test	IEC 60601 Test level	Compliance level	Electromagnetic environment – guidance
Electro-static discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air	± 6 kV contact ± 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.

Electrical fast transients/ burst IEC 61000-4-4	± 2 kV for power line supplies ± 1 kV for input/output lines	Not applicable *) Not applicable *)	Not applicable *)
Surge IEC 61000-4-5	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth	Not applicable *) Not applicable *)	Not applicable *)

<p>Voltage dips, short inter-rup-tions and volt-age variations on power sup-ply input lines IEC 61000-4-11</p>	<p><5 % U_T (>95 % dip in U_T) for 0.5 cycle</p> <p>40 % U_T (60 % dip in U_T) for 5 cycles</p> <p>70 % U_T (30 % dip in U_T) for 25 cycles</p> <p><5 % U_T (>95 % dip in U_T) for 5 s</p>	<p>Not applicable *)</p>	<p>Not applicable *)</p>
--	--	------------------------------	------------------------------

Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment
---	-------	-------	--

NOTE U_T is the a.c. mains voltage prior to the application of the test level.

*) *Battery powered equipment*

Electromagnetic immunity – cont.

The COM-DEX is intended for use in the electromagnetic environment specified below. The customer or the user of a COM-DEX should assure that it is used in such an environment.

Immunity Test	IEC 60601 Test level	Compliance level	Electromagnetic environment – guidance
			Portable and mobile RF communications equipment should be used no closer to any part of the COM-DEX, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.

			Recommended separation distance
Conducted RF	3 Vrms	3 Vrms	$d = 1.2 \sqrt{P}$
IEC 61000-4-6	150 kHz to 80 MHz		
Radiated RF	3 V/m	3 V/m	$d = 1.2 \sqrt{P}$ 80 MHz to 800 MHz
IEC 61000-4-3	80 MHz to 2.5 GHz		$d = 2.3 \sqrt{P}$ 800 MHz to 2.5 GHz

Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).

Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey ^a, should be less than the compliance level in each frequency range ^b.

Interference may occur in the vicinity of equipment marked with the following symbol:

SYMBOLS

Symbols commonly used by WSAUD A/S in medical device labeling (labels/IFU/etc.)

Symbol	Title/Description
	Manufacturer The product is produced by the manufacturer whose name and address are stated next to the symbol. If appropriate, the date of manufacture may also be stated.
	Date of manufacture The date when the product was manufactured.
	Batch code The product's batch code (lot or batch identification).
	Catalog number The product's catalog (item) number.
	Serial number The product's serial number.*
	Consult instructions for use The user instructions contain important cautionary information (warnings/precautions) and must be read before using the product.

Symbol	Title/Description
	<p>Caution/Warning Text marked with a caution/warning symbol must be read before using the product.</p>
	<p>WEEE mark “Not for general waste” When the product is to be discarded, it must be sent to a designated collection point for recycling and recovery.</p>
	<p>CE mark The product is in conformity with the requirements set out in European CE marking directives.</p>
	<p>RCM mark The product complies with EMC and radio spectrum regulatory requirements for products supplied to the Australian or New Zealand market.</p>
	<p>Interference Electromagnetic interference may occur in the vicinity of the product.</p>

*The six- or seven-digit number on the product is the serial number. Serial numbers may not always be preceded by **SN**

FCC AND ISED STATEMENTS

FCC ID: TTY-CMDEX

IC: 5676B-CMDEX

Federal Communications Commission Statement

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation.

NOTE:

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation.

This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this

equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

NOTE:

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

Changes or modifications to the equipment not expressly approved by WSAUD A/S could void the user's authority to operate the equipment.

ISED STATEMENT:

This device contains licence-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s). Operation is subject to the following two conditions:

1. This device may not cause interference.
2. This device must accept any interference, including interference that may cause undesired operation of the device.

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ISED RADIATION EXPOSURE STATEMENT:

This equipment complies with ISED RF radiation exposure limits set forth for an uncontrolled environment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

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